## User Instruction Manual

Hoyer® Stature

To avoid injury, read user manual prior to use.

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
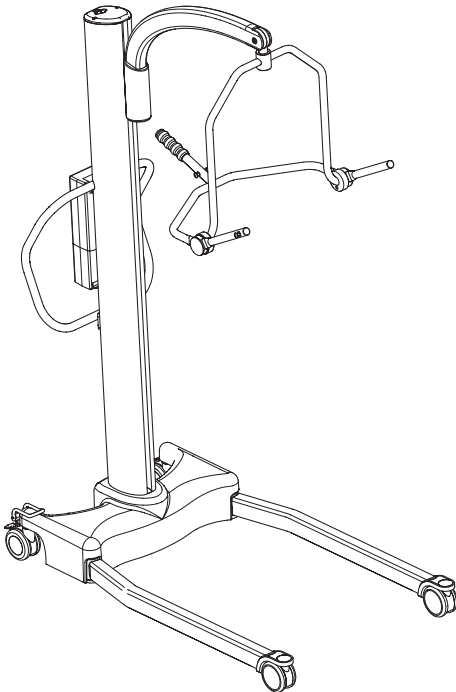
## Contents

1. Hoyer Stature Patient Lift .....	3
2. Introduction: About Your Lift.....	4
3. Assembly and Commissioning Instructions.....	5
4. Safety Precautions .....	9
5. Standard Control Box & Handset.....	11
6. Smart Monitor & Handset .....	12
7. Operating Instructions .....	16
8. Removal of Spreader Bar/Cradle Systems .....	20
9. Charging Instructions.....	22
10. Maintenance Schedule & Daily Check List .....	23
11. Technical Specifications .....	26
12. Warranty .....	29
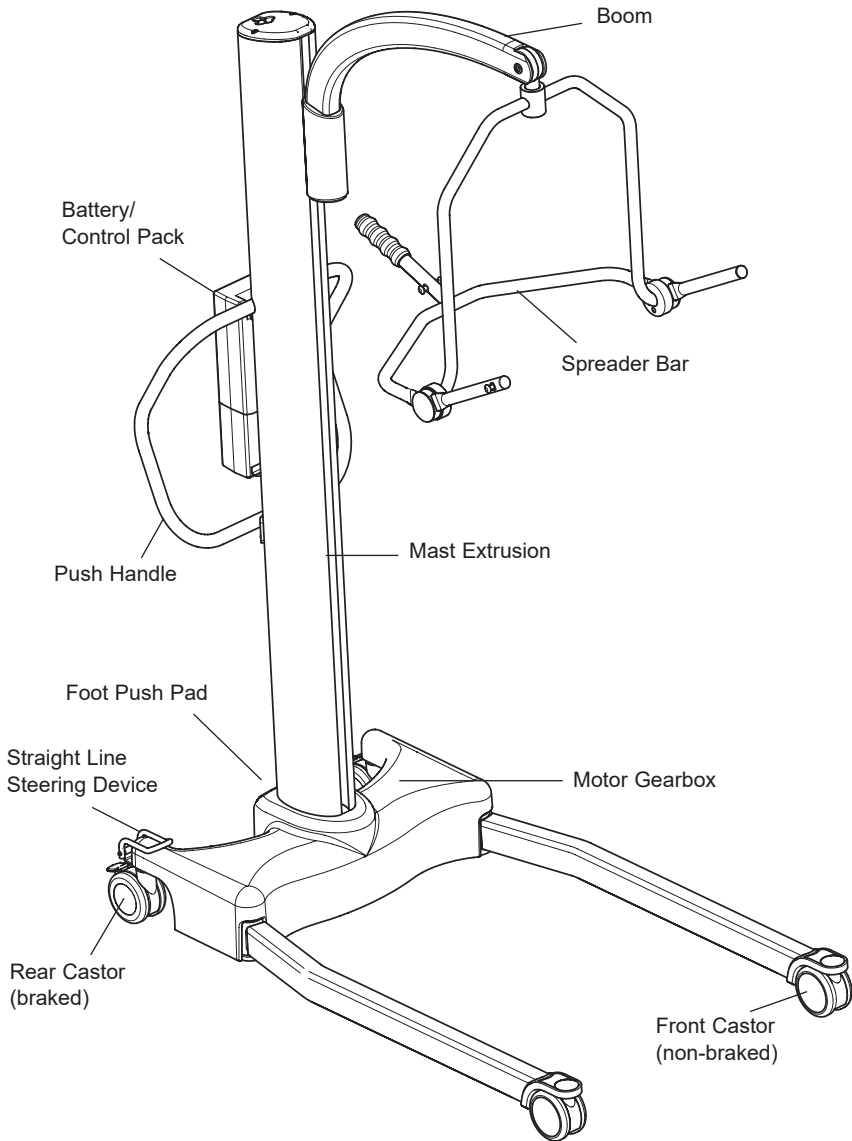
# 1. Hoyer Stature Patient Lift

**READ AND UNDERSTAND THE INSTRUCTIONS IN THIS MANUAL  
BEFORE USING YOUR STATURE**

## 2. Introduction: About Your Lift

Each Stature lift is fully assembled, load tested and certified before being packed/shipped.

The packing consists of a strong, purpose built carton that is used for both export and domestic markets to ensure the safe arrival of the lift. A number of documents are supplied in a wallet, and packed with each lift and should be kept safely for future reference.

- TEST CERTIFICATE
- USER MANUAL

The TEST CERTIFICATE is an important document and should be kept for reference purposes.

To properly maintain your lift please refer to the maintenance schedule included in this document. If you are at all unsure what your country's servicing requirements are, please check with your dealer and/or a local government agency.

### Statement of Intended Use

The intended use of this lifting device is for the safe lifting and transfer of an individual from one resting surface to another (such as a bed to a wheelchair). Joerns Healthcare recommends that the transfer of a patient is fully risk assessed and conducted safely over a short distance only.

The Hoyer Stature is suitable for patients in the SITTING, SITTING/RECUMBENT and RECUMBENT positions.

The Hoyer Stature is an electrically operated patient lift, designed to support and promote safe patient handling and transfer for both the patient and carer.

The Hoyer Stature lift incorporates a 4-point cradle as standard and is designed to be used in conjunction with the Hoyer range of slings. A 6-point cradle option is available as an add-on. The examples of slings suitable for use with this device are listed as follows:

- Hoyer Quickfit
- Hoyer Access
- Hoyer Full Back
- Hoyer Comfort
- Hoyer Quickfit Deluxe
- Hoyer Long Seat

### Expected Service Life

Hoyer Professional lifts are designed and tested for a minimum service life of ten (10) years, subject to the use and maintenance procedures stated in this manual. Use, other than in accordance with these instructions, may compromise service life.

## 3. Assembly and Commissioning Instructions

### Carton Contents

Place the carton in a clear working area and open carefully. The carton contains:

- HOYER STATURE LIFT
- WALLET CONTAINING DOCUMENTS
- HAND CONTROL
- BATTERY PACK
- DESK TOP CHARGER/STAND/CORD
- FASTENER KIT (INCLUDES SCREWS, ALLEN KEY, HEX SOCKET, CAPPING PLATE)

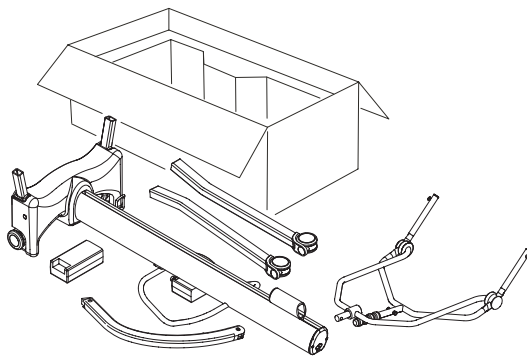
### **⚠ WARNING**

**The Hoyer Stature is heavy and will need to be lifted with care. You may need assistance to lift the Hoyer Stature from the carton.**

### Assembly Instructions


Remove all the parts from the carton and place on the floor, taking care to protect the finish from damage.

1. Place the chassis and mast (which are connected) in a clear space. The mast and chassis should be rested on the push handle, which will ensure the product remains stable while connecting the legs and boom to the product.

2. Fit the legs to the chassis using two 6mm countersunk screws per leg as provided.

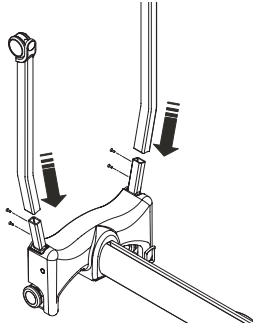
**NOTE:** There is a left and a right leg and they are configured as shown below


3. Using Allen Key provided, tighten the 6mm screws located on each side of the chassis.

**⚠ WARNING**

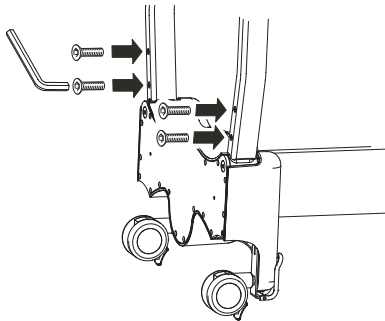
The screws should be secured firmly.

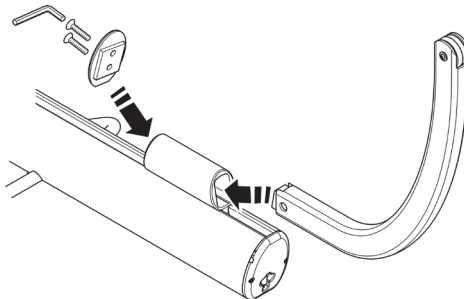4. Insert the curved boom section into the boom carriage. Secure the assembly by inserting the two 6mm countersunk screws into the blue capping plate provided in the fastener kit, as shown below and tighten firmly.


5. Attach four point positioning cradle to the lift boom by inserting the split pin. Ensure pin is secure.

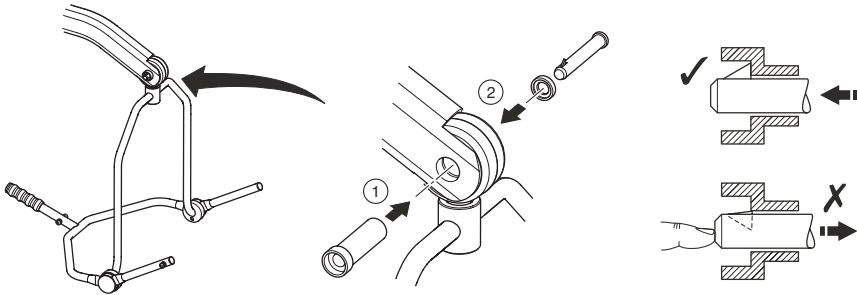6. Stand the Stature upright and apply the brakes.

### **⚠ WARNING**

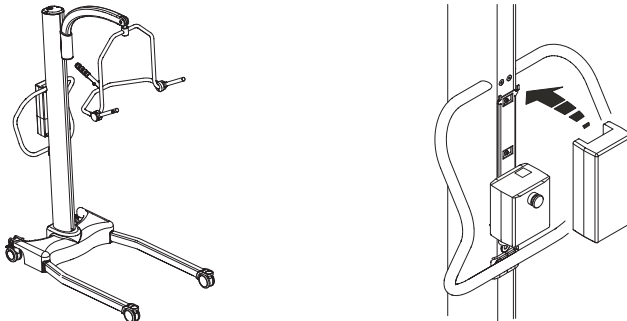Be careful when lifting the Stature to the upright position, as it is heavy.


## Disassembly

The lift should not be disassembled unless for service, repair or transport if necessary. If disassembly is necessary, follow the assembly instructions in reverse sequence.


7. Fit power pack to the lift and make sure the latch holding the pack in place is fully engaged. "Click" in place.
8. Plug the handset and two cables from the mast into the bottom of the controller. Refer to the drawing on page 8 for correct positions.

**ALWAYS CHECK THE FOLLOWING BEFORE OPERATION**

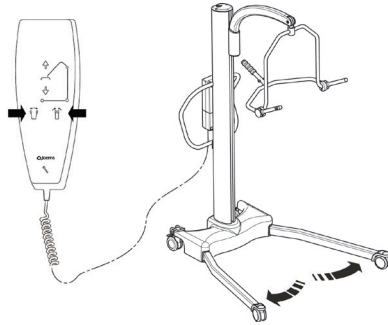
- Push the open and close buttons on the hand control to confirm that the legs of the lift open and close satisfactorily.

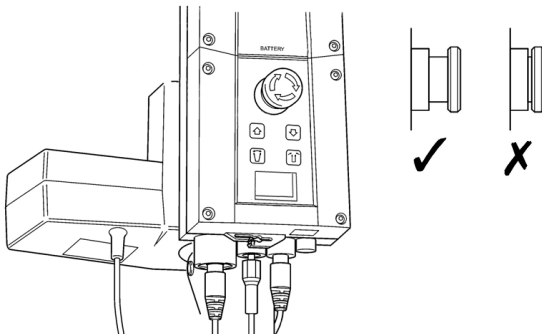- The red emergency stop button, located on the rear of the control box, is in the OFF (out) position.

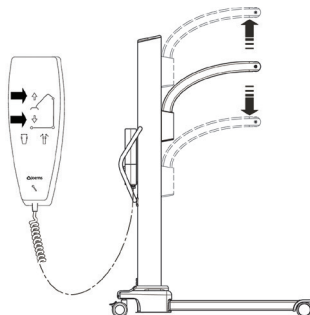- Push the up and down buttons on the hand control to confirm that the boom rises and lowers satisfactorily.

## 4. Safety Precautions

Please read and follow the safety precautions listed below. The operation and use of Hoyer patient lifts is simple and straightforward. Following these few basic safety precautions will make lifting operations easy and trouble free.

### **READ AND UNDERSTAND THE USER INSTRUCTION MANUAL BEFORE USING THE HOYER STATURE LIFT**

**WARNING:** Important safety information for hazards that might cause serious injury.

**CAUTION:** Information for preventing damage to the product.

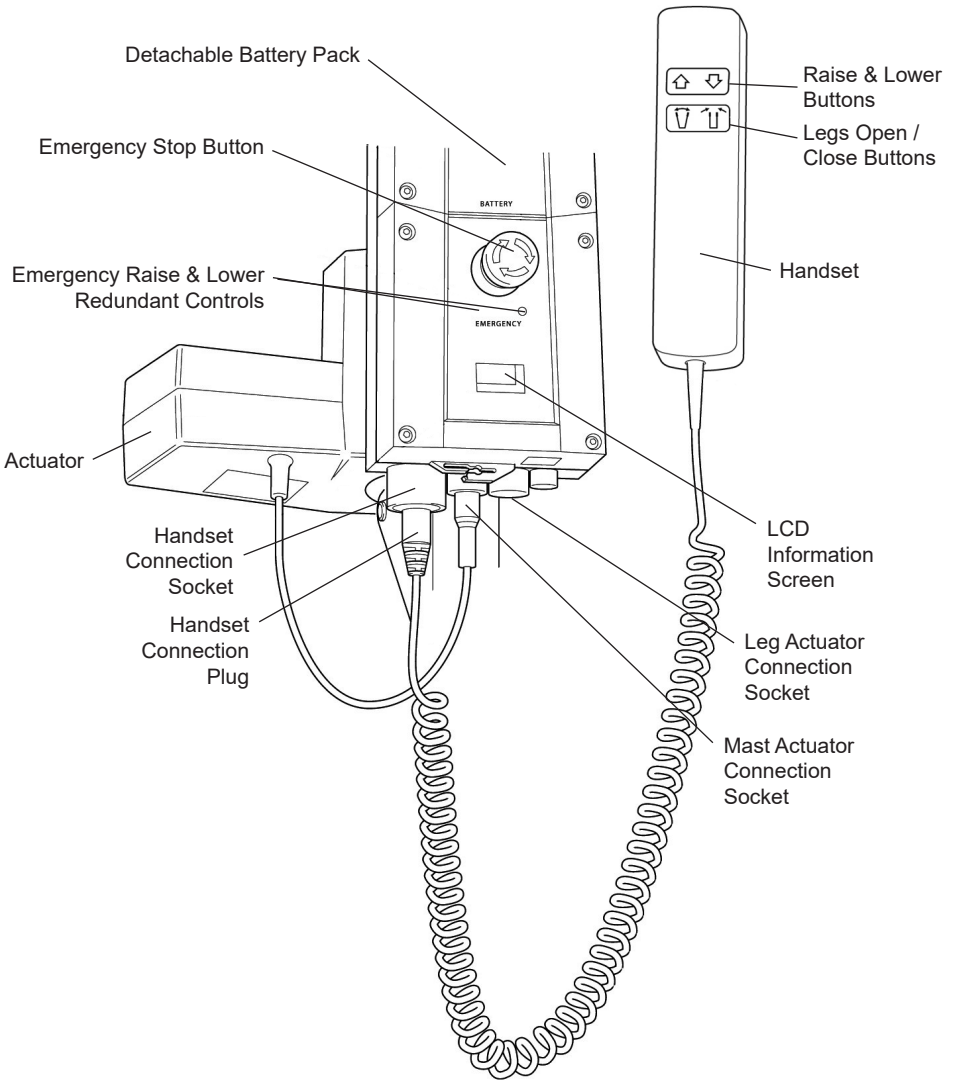
**NOTE:** Information to which you should pay special attention.

### **WARNING**


- **DO NOT lift a patient unless you are trained and competent to do so.**
- **ALWAYS carry out the DAILY CHECK LIST located towards the end of the manual before using the lift.**
- **DO NOT exceed the safe working load of the lift.**
- **ALWAYS plan your lifting operations before commencing.**
- **ALWAYS familiarize yourself with the operating control and safety features of a lift before lifting a patient.**
- **DO NOT use a sling unless it is recommended for use with the lift.**
- **ALWAYS check the sling is suitable for the particular patient and is of the correct size and capacity.**
- **NEVER use a sling, which is frayed or damaged.**
- **ALWAYS fit the sling according to the instructions provided (user instructions).**
- **ALWAYS check the safe working load of the lift is suitable for the weight of the patient.**
- **ALWAYS carry out lifting operations according to the instructions in the user manual.**
- **NEVER disconnect or bypass a control or safety feature because it seems easier to operate the lift.**
- **ALWAYS lock the wheels when lifting from the floor.**
- **DO NOT attempt to manoeuvre the lift by pushing on the mast, boom or patient.**
- **ALWAYS manoeuvre the lift with the handle provided. A foot push pad is also provided.**
- **DO NOT push a loaded lift at speeds, which exceed a slow walking pace (2.6 ft/sec).**
- **DO NOT push the lift over uneven or rough ground. Particularly if loaded.**
- **DO NOT attempt to push/pull a loaded lift over a floor obstruction.**

- **NEVER** force an operating/safety control. All controls are easy to use and do not require excessive force.
- **DO NOT** park a loaded lift on **ANY** sloping surface.
- **DO NOT** use electric lifts in a shower.
- **DO NOT** use or store a lift in a wet or corrosive environment such as shower, bath or pool locations.
- **DO NOT** charge batteries in a bathroom or shower room.
- **DO NOT** place or store batteries under direct sunlight or near a heat source.
- **YOUR** lift is for patient lifting. **DO NOT** use it, or allow it to be used, for any other purpose.
- **DO NOT** bump the lift down steps, loaded or unloaded.
- **DO NOT** attempt to negotiate a loaded lift on a slope, which exceeds 1:12 (approximately 5 degrees)
- **DO NOT** attempt to negotiate a slope without a second helper being present.
- **DO NOT** use a loaded lift on thick pile carpet.
- **NEVER** operate the lift with loose or missing parts or fasteners.
- **INSPECT** all precautionary labels on the lift. Order and replace all labels that cannot be easily read.

## 5. Standard Control Box & Handset

## 6. Smart Monitor & Handset
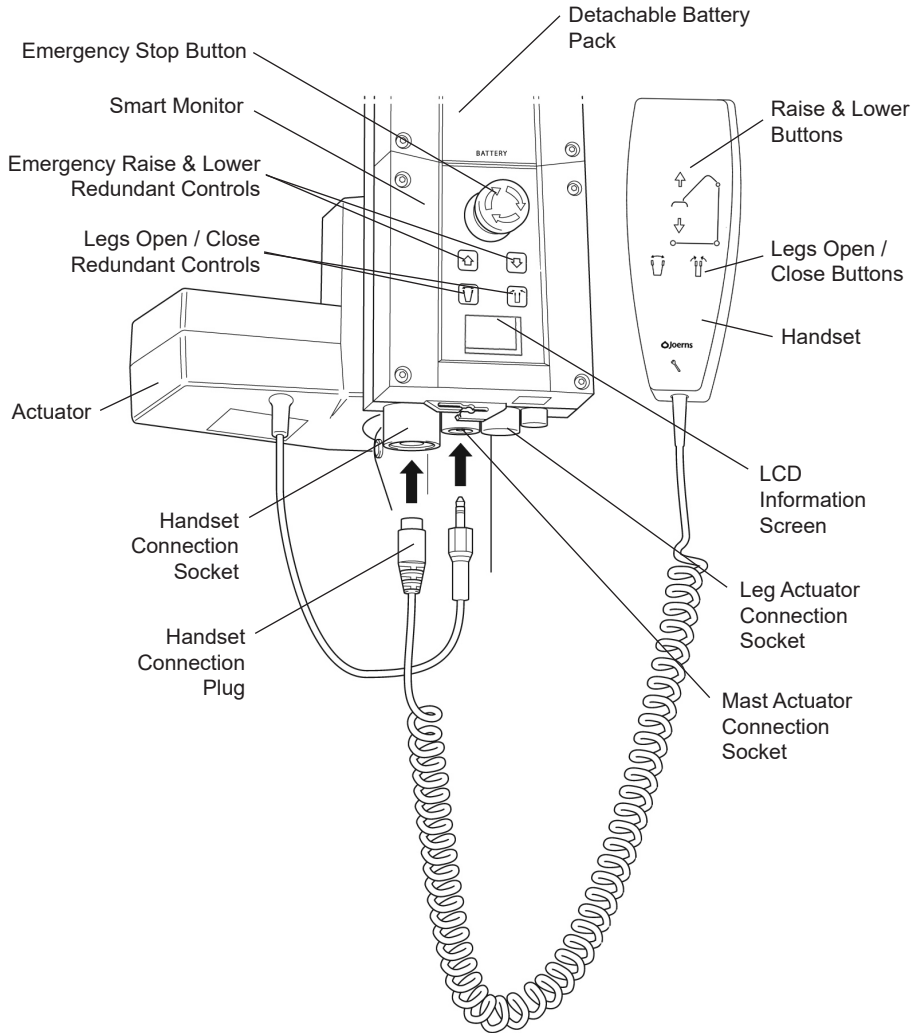
## Intended Use

The Hoyer Smart Monitor is a control system for Hoyer lifts. The Smart Monitor stores useful servicing information about the lift that can be recalled when required. This servicing information includes:

- Number of patient lift cycles
- Total work done by actuator
- Number of lift overloads (attempted lifts above the safe working load)
- Number of days since last service interval

The Hoyer Smart Monitor contains a microprocessor inside making it possible to read out service data via the on-board LCD screen.

## Features

- Data collection in the control box with data display via LCD screen
- Work counter
- Intelligent cycle counter
- Service indicator
- Service interval indicator
- Overload information
- 3 step battery indicator

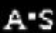
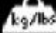
## Benefits

- Improved safety for both patient and carer
- Accurate service data available at the touch of a button
- Optimised product life time
- Ease of maintenance for engineers and service technicians

## LCD Display Screen

The Hoyer Smart Monitor has the option to read out information via the LCD display screen. It is possible to read out total lifting cycles, total work done, overloads and number of days since last service, which can be used to quickly and easily evaluate the condition of the lift actuator.

This information is accessed by a ½ second press on the 'UP' button on the User handset or redundant 'UP' button on the Smart Monitor.

	12034	Total Lifting Cycles
	1257000	Total Work Done (Amps * Seconds)
	7	Number of Overloads
	90/360	Number of Days (since last service) / Number of Days (between services)

## Service & Usage Information

When it is time for a service, the service symbol will appear on the display. In addition, when the user handset is activated, the Smart Monitor will give an audible signal giving notice to users that a PM check is recommended (see recommended maintenance schedule later in this manual).

The service symbol will be displayed each time the raise or lower keys are depressed on the user handset.

To reset the service interval to one year, depress the leg open and leg close keys simultaneously for five seconds.

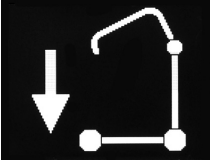
If the lift stops because of an overload (an attempt to lift more than the safe working load) the overload symbol will appear on the LCD display and the lift will cease. Lower the boom and remove excess weight.


### WARNING

If the LCD display screen on your Smart Monitor has recorded an overload, Joerns Healthcare recommend that **ALL** routine daily checks are carried out on the lift prior to further use (for a list of daily checks, please refer to your lift user manual). Joerns Healthcare also recommend that you contact your authorised Hoyer service provider for additional guidance.

## On-Board Redundant Controls

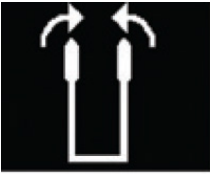On-board redundant controls enable the lift to be raised or lowered in the event of an emergency.

If your lift incorporates powered leg positioning, there are redundant controls to both open and close the legs.


## Battery Information

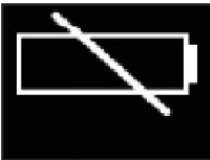The display showing full battery means that the battery is fully charged and the lift is ready for use.

The display showing a half empty battery indicates that it is time to charge the battery.

The empty battery symbol showing on the display indicates that the battery has no capacity left and should be placed on charge immediately.

### **⚠ WARNING**

To avoid possible permanent damage to the battery, the battery should be placed on charge as soon as the display indicates the half empty battery symbol.

## 7. Operating Instructions

### 1. Leg adjustment

The legs on the Hoyer Stature are electrically adjustable for width. The legs can be opened to enable access around armchairs or wheelchairs. For transferring and negotiating narrow doorways and passages the lift legs should be in the closed position.

Electric leg adjustment is achieved by pressing the appropriate buttons on the hand control. The legs will be locked whenever the hand control switch is released.

### 2. Castors and Braking

The lift has two braked castors which can be applied for parking. When lifting, the castors should be left free and un-braked. The lift will then be able to move to the centre of gravity. If the brakes are applied it is the patient that will swing to the centre of gravity and this may prove disconcerting and uncomfortable. Attached to the rear castor is a straight line steering device, which when applied to the castor will facilitate straight line movement.

### 3. Raising and lowering the boom

The movement of the boom carriage is achieved by a powerful motor gearbox, which is controlled by a simple hand control unit. The hand control has two buttons with directional arrows UP and DOWN ↑↓. The motor gearbox stops automatically at the limit of travel in both directions. The hand control plugs into a socket at the base of the control box.

**SAFETY NOTE:** The downward movement of the boom will stop if the lift senses that the patient has encountered an obstruction.

### 4. Emergency Stop

The red Emergency Stop Button is located on the front of the control box and is activated by pressing in. This will cut all power to the lift and only be reset by twisting the button anticlockwise and releasing.

### 5. Redundant controls

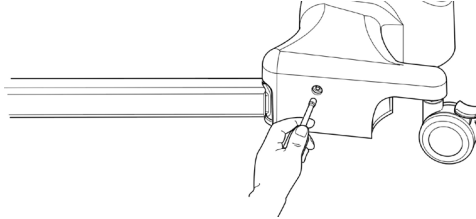
All Hoyer lifts are fitted with raise and lower buttons on the control box. These are located underneath the emergency stop button and can be used to lower/raise the patient should the hand control fail.

## 6. Mechanical Emergency Down


In the case of a complete electrical failure the Stature lift is fitted with mechanical lowering device. This device is located on the left hand side of the lift's chassis.

The lowering mechanism is operated by inserting a 6mm Hex socket into the aperture and onto the mechanism turning clockwise to raise the lift and anticlockwise to lower the lift. This can be done manually with a socket wrench.

### RECOMMENDATION:

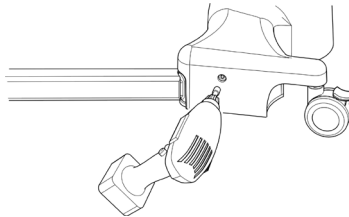In order to raise or lower the boom more quickly it is recommended that a 6mm Hex socket be used in conjunction with a 1/4 inch drive adaptor and an electric reversible drill (NOTE: do not use hammer setting). This will increase the speed in which the patient can be safely raised and lowered. A 6mm Hex socket is contained in the top cap located on the top of the mast.

**NOTE:** Before raising or lowering the patient, ensure you have planned your destination carefully. For instance back onto a chair or bed.

## 7. Batteries

The batteries are protected from deep discharge by a LOW VOLTAGE ALARM. This will sound when the batteries need recharging and the hand control is being operated. It will not sound independently of the hand control being operated. **DO NOT IGNORE THIS WARNING ALARM.** Complete the lifting operation and place the battery on charge (see charging instructions).

## 8. Slings

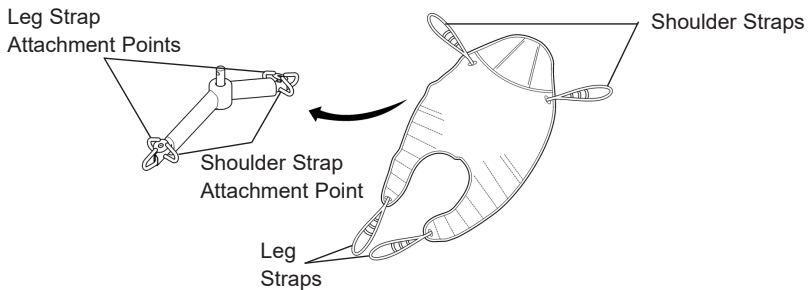
The Hoyer Stature has the option of both a 6 point spreader bar and a 4 point positioning cradle. Both systems use different attachment methods and slings.

The 6 point system uses slings with webbing loops, which allow positioning adjustment by selecting different coloured loops. The 4 point cradle uses the Securifi safety clip system and adjustment is made by rotating the 4 point cradle either upward or down.

**Loop strap slings:** Hoyer Quickfit, Full Back, Quickfit Deluxe, Long Seat, Access and Silkfit.

**Safety clip slings:** Hoyer Comfort


The 6 point system attaches to the spreader bar hooks via loop straps (shown below), each sling is supplied with instructions. Please study the instruction guide before use.

### 6 Point spreader bar


Example of 6 point spreader bar sling with loop attachments.

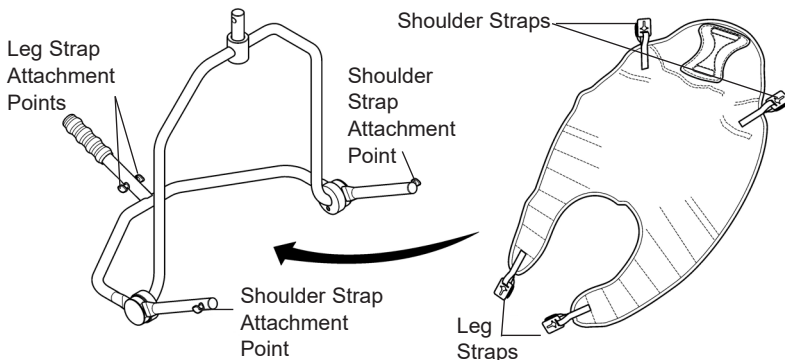The four point cradle sling attaches to the cradle studs via a safety clip system (shown below), each sling is supplied with instructions. Please study the instruction guide before use.

### 4 point positioning cradle

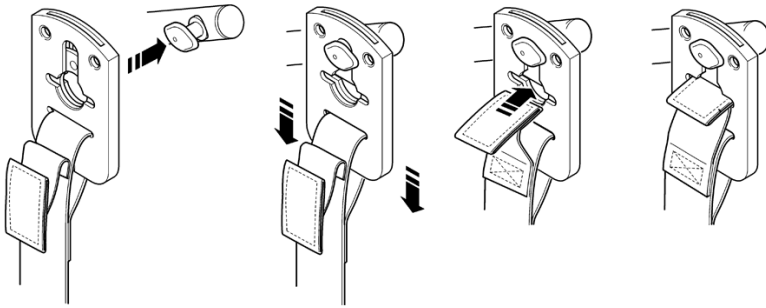Example of 4 point cradle sling with safety clip system.

### Securi3 sling attachment system

Example of installation.

**NOTE:** For detailed fitting instructions, please refer to the user guide supplied with each sling.

### **⚠ WARNING**

Joerns Healthcare recommends that slings be checked regularly and particularly before use for signs of fraying or damage. **DO NOT** use slings that are worn or damaged.

### **⚠ WARNING**

**HOYER RECOMMENDS THE USE OF GENUINE HOYER PARTS.** Hoyer slings and lifters are not designed to be interchangeable with other manufacturer's products. Using other manufacturer's products on Hoyer products is potentially unsafe and could result in serious injury to patient and/or caregiver.

### **⚠ WARNING**

Refer to maximum weight capacity of lift. Sling capacity is limited by the maximum capacity of the lift.

## 8. Removal of Spreader Bar/Cradle Systems

### FOLLOW THESE INSTRUCTIONS EXACTLY

Your Stature lift has the option of using both a 6 point spreader bar and 4 point positioning cradle. The removal of either of these systems is completed easily by the use of a quick-release pin. This is done quickly and easily, but you must read the following safety instructions to ensure the spreader bar/cradle is safely locked into position before use.

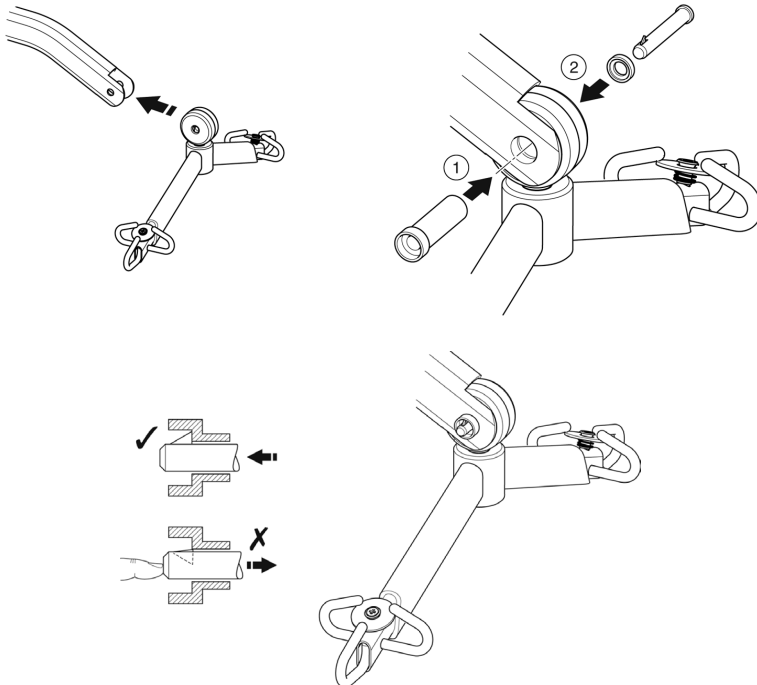
### Connection

To connect, raise the spreader bar/cradle up toward the boom slot.

Insert the pin sleeve through the plastic end cap and to support the spreader bar/cradle.

Once aligned in the boom slot reinsert the quick release pin fully into the sleeve until it locks into position. **YOU MUST CHECK THAT THE PIN IS SECURELY FITTED.**

To do this, press your finger into the barbed end. If the pin remains stationary the lift is safe to use.


## Removal

**CAUTION:** Be sure to support the weight of the spreader bar/cradle before removing pin.

To detach pin, depress the spring barb (using a flat blade electrical screw driver or similar) and pull the pin out.

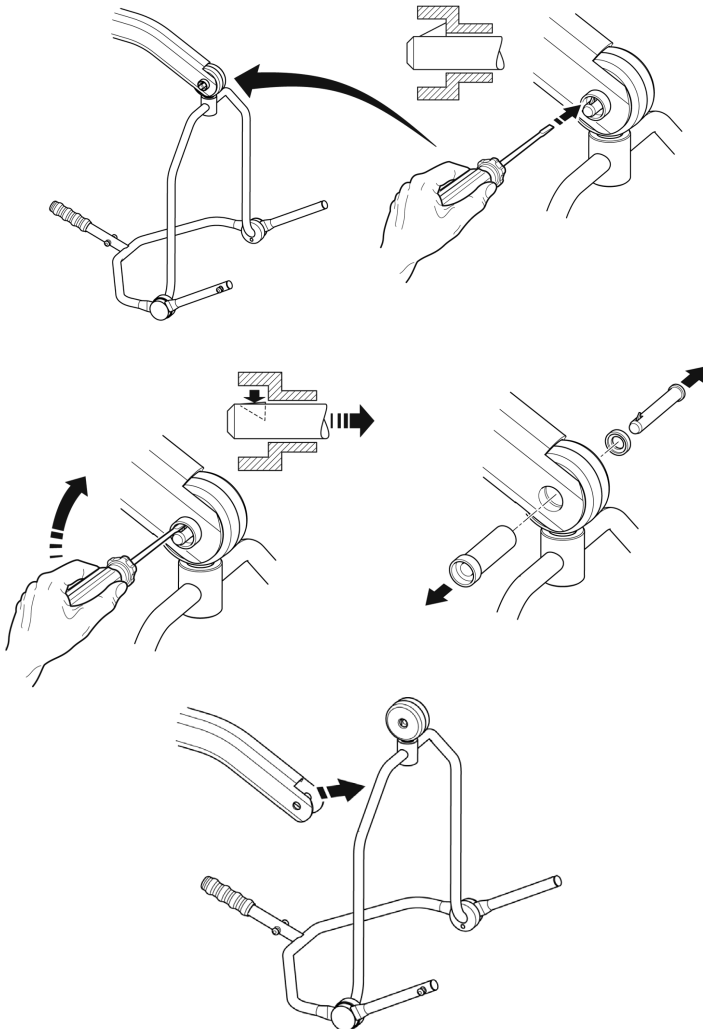Support the spreader bar/cradle, and then pull the pin sleeve in the opposite direction, removing it completely from the boom. This will release the spreader bar completely.

## 9. Charging Instructions

When the power pack needs charging it is removed from the lift and fitted to an off board charging unit.

1. Remove the power pack from the lift. The pack is retained by a simple latch at the top of the power pack. Lift the latch and the power pack will be released.
2. Fit the power pack to the charging unit. The location and latching of the power pack to the charger is the same system as used on the lift.
3. Plug the charger mains plug into a suitable mains outlet and switch the mains supply ON.
4. Charging is automatic and will fully charge the batteries over a period of four (4) hours.

**Note:** Even if the charger is left plugged in for extended periods it will not allow the batteries to overcharge.

- a) Green Light - Indicates main power is on.
- b) Yellow Light - Indicates battery is charging.
- c) Battery will be fully charged when yellow light goes off.

**Note:** It is recommended that the battery be charged immediately upon receipt.

5. To return the lift to service, unplug the charger and remove the power pack from the charger. Fit the power pack to the lift and make sure the latch holding the pack in place is fully engaged. "Click" in place. The charging of Hoyer electric lifts is simple and straightforward, but it is important to follow the charging instructions closely. Please pay particular attention to the following points, they will help you avoid problems with discharged batteries.

### **WARNING**

- **The battery, charger, hand control and control box are NOT to be opened by unauthorised personnel. (Contact your distributor for warranty and repairs).**
- **DO NOT touch battery/charger terminals.**
- **KEEP the batteries fully charged. Place the battery on charge whenever it is not in use. If it is more convenient to do so, place on charge every night. The charger will not allow the batteries to overcharge.**
- **NEVER run the batteries completely flat. As soon as the audible warning sounds, complete the lifting operation in hand and place on charge.**
- **To avoid possible permanent damage to the battery, the battery should be placed on charge as soon as the display indicates the half empty battery symbol.**
- **NEVER store the battery for long periods without regular charging throughout the storage period.**
- **NEVER leave the battery pack connected with the charger unit unplugged.**
- **DO NOT leave the charger plugged in with no battery connected.**
- **NEVER disconnect the charger plug by pulling on the cable.**
- **BE CAREFUL not to trip over the charge lead.**
- **DO NOT charge batteries in a bathroom or shower room.**

## 10. Maintenance Schedule & Daily Checklist

All Hoyer products are designed for minimum maintenance, however some safety checks and procedures are required. A schedule of DAILY tasks are detailed below. Daily checks and a yearly service, inspection and test will ensure a lift is kept in optimum safe working condition. A list of spare parts is available upon request.

DAILY CHECK LIST: Joerns Healthcare strongly recommends the following checks be carried out on a daily basis and before using lift.

- MAKE sure the lift moves freely on its castors.
- MAKE sure the spreader bar is free to rotate and swing. Check the spreader bar is firmly attached to the boom.
- ENSURE the mast is fully engaged into the mast slot and the mast locking device is securely tightened.
- EXAMINE the sling hooks on the spreader bar and side suspenders for excessive wear. If in doubt - do not use.
- MAKE sure the legs open and close correctly.
- OPERATE the hand control to confirm the boom raises and lowers satisfactorily.
- CONFIRM the lift is not giving a low battery indicator when the hand control is operated. If a low battery is indicated, DO NOT use and place on charge immediately.
- Check the operation of the emergency stop button.
- EXAMINE slings for fraying or other damage. DO NOT use any sling if damaged or if the sling shows signs of wear.

### MAINTENANCE

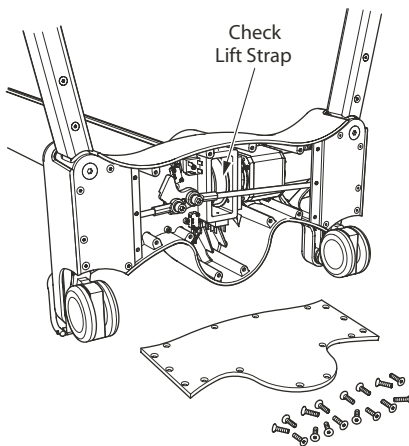
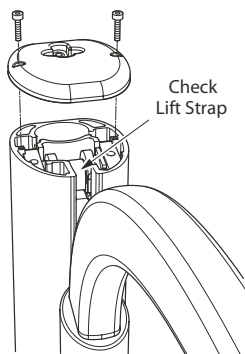
Joerns Healthcare recommends regular inspection and maintenance. Please refer to the chart on the next page.

**\*THESE CHECKS SHOULD INCLUDE:**

✓ = Recommended

	Initially	Before Use	Annual Service
<b>1. SPREADER BAR:</b> Check the spreader bar for freedom of rotation and swing. Check for wear on the central pivot. Check for firm attachment to the boom.	✓	✓	✓
<b>2. BOOM:</b> Check the attachment of the boom to the mast carriage. Make sure there is only minimal movement.	✓	✓	✓
<b>3. MAST:</b> Check it is firmly connected to the chassis.	✓	✓	✓
<b>4. MOTOR GEARBOX:</b> This should require no maintenance other than checking for correct operation and listening for unusual noise.	✓		✓
<b>5. CONTROL BOX/SMART MONITOR:</b> Check the function of the emergency stop button. Inspect the hand control socket for correct fitting. Check functioning of the hand control. Check the redundant controls and confirm they operate as intended.	✓	✓	✓
<b>6. BATTERIES:</b> The batteries are housed in the power pack and should not require maintenance other than the regular charging as detailed in the charging instructions. Check that the connections remain clean.			✓
<b>7. LEG ADJUSTMENT:</b> Check the legs operate in both full extensions (inward/outward).	✓		✓
<b>8. CASTORS:</b> Check all castors for firm attachment to the legs. Check for free rotation of the castor and the wheels.	✓	✓	✓
<b>9. CLEANING:</b> Clean with ordinary soap and water and/or any hard surface disinfectant. Harsh chemical cleaners or abrasives should be avoided as these may damage the surface finish of the lift. Avoid wetting any of the electrical parts.		✓	✓
<b>10. BASE AND WHEELS:</b> Ensure base is even and level (all four wheels are on the floor).	✓	✓	✓
<b>11. SLINGS:</b> Check for wear and fraying.	✓	✓	✓
<b>12. LUBRICATION:</b> Lubricate pivot joints with a silicone based oil, including mast and boom connections, pedal assembly, spreader bar joint (only if required),			✓
<b>13. HAND SET:</b> Ensure plugged fully into controller.	✓	✓	
<b>14. QUICK RELEASE PIN:</b> Ensure the pin is securely fitted before lifting.	✓	✓	
<b>15. HARDWARE:</b> Check all nuts, bolts, screws and fasteners for excessive wear and for tightness. Replace as required.		✓	✓
<b>16. LIFTING STRAP:</b> Your lift strap should be checked at service intervals for any sign of wear (ideally every six months). In order to view the strap it is recommended to remove the cap at the top of the mast, cycle the lift in both directions whilst observing the condition of the strap. If you see any wear (fraying, creasing or similar) it is recommended an authorised, Joerns Healthcare Dealer or qualified, competent service engineer replaces the strap IMMEDIATELY. For details regarding strap maintenance and replacement please request our detailed service guide. (See below for details)	✓		✓

## 11. Technical Specifications

Safe Working Load .....	500 lbs.....	227 kgs
Maximum Overall Length .....	54.7 inches.....	1390 mm
Minimum Overall Length .....	50.8 inches.....	1290 mm
Maximum Overall Height.....	83.8 inches.....	2130 mm
Minimum Overall Height.....	76.6 inches.....	1945 mm
Height at Maximum Reach.....	77.8 inches.....	1975 mm
Reach at Maximum Height.....	24.6 inches.....	625 mm
Reach at Minimum Height.....	24.6 inches.....	625 mm
Maximum Reach* .....	24.6 inches.....	625 mm
Turning Radius .....	61.0 inches.....	1550 mm
Legs Open - External Width .....	53.5 inches.....	1360 mm
Legs Open - Internal Width .....	48.4 inches.....	1230 mm
Legs Closed - External Width .....	29.9 inches.....	760 mm
Legs Closed - Internal Width .....	25.1 inches.....	640 mm
Overall Height of Legs.....	4.3 inches.....	110 mm
Ground Clearance .....	1.2 inches.....	30 mm
Front Twin Castors .....	4 inches.....	100 mm
Rear Braked Castors.....	4 inches.....	100 mm

\* Reach = centre of spreader bar to the front of the mast

### 4 Point Position Cradle (measurement to top of location pins)

**NOTE:** 4 point positioning cradle is standard fitment with the Stature lift.

Cradle Max. Height.....	61 inches.....	1550 mm
Cradle Min. Height (usable) .....	15.4 inches.....	390 mm

### 6 Point Spreader Bar (measurement to top of location pins)

**NOTE:** 6 point positioning cradle is optional for the Stature lift.

Spreader Bar Max. Height.....	77.7 inches.....	1975 mm
Spreader Bar Min. Height (usable).....	26.6 inches.....	675 mm

### Weights

Mast, Base & Boom Assembly .....	142 lbs.....	64.5 kgs
4 point positioning cradle .....	13.2 lbs.....	6 kgs
Power Pack .....	6.6 lbs.....	3 kgs
Total .....	162 lbs.....	73.5 kgs

**NOTE:** Manufacturing tolerances apply to all dimensions.

**ELECTRICAL SPECIFICATIONS:**

BATTERIES.....	24 volt Rechargeable sealed lead acid type
BATTERY CAPACITY .....	3.2 Ampere hours
CHARGER RATED INPUT .....	100 - 240 V AC/ 50/60 Hz
CHARGER RATED OUTPUT .....	29.5 VDC, Max. 19 W

**ELECTRIC SHOCK PROTECTION**

CHARGER.....	CLASS II (EN 60601-1)
LIFT .....	INTERNAL POWER SOURCE

**DEGREE OF SHOCK PROTECTION**

CHARGER.....	TYPE B (EN 60601-1)
LIFT .....	TYPE B (EN 60601-1)

**ENVIRONMENTAL CONDITIONS:**

Outside this environment functionality and safety may be compromised.

**OPERATING**

TEMPERATURE .....	5°C to 40°C
RELATIVE HUMIDITY.....	20% to 90% @ 30°C - not condensing
ATMOSPHERIC PRESSURE.....	700 to 1060 hPa

**STORAGE**

TEMPERATURE .....	-10°C to +50°C
RELATIVE HUMIDITY.....	20% to 90% @ 30°C - not condensing
ATMOSPHERIC PRESSURE.....	700 to 1060 hPa

**IP RATINGS**

CONTROL BOX .....	IPX4
ACTUATOR .....	IPX4
OFF BOARD CHARGER .....	IPX5
BATTERY .....	IPX5
HANDSET .....	IPX5

**DUTY CYCLES**

ACTUATOR.....	10% (2 min./18 min.)
CHARGER .....	Approx. 4 hrs.

**ACOUSTICS**

A-Weighted sound power level .....	60 dB (A)
------------------------------------	-----------

## KEY SYMBOLS:

The following symbols are used on the charger, control unit and battery:

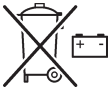
Type B equipment, as per EN 60601-1

Class 2 equipment

The disposal of the charging and control unit should not be mixed with general household waste

The disposal of batteries should not be mixed with general household waste.

The disposal of electronics should not be mixed with general household waste

For indoor use

ATTENTION, consult accompanying documents.

WEEE Producers registration number WEE/GG0464RZ

## 12. Warranty

This warranty covers HoyerPro lifts only. Lifts not covered under this warranty include, but are not limited to Advance-H, HML400, HPL600WB, HPL600WBSC, HPL402, C-HLA (and variations).

HoyerPro lifts are guaranteed for a period of two years from the date of delivery against defects in materials and workmanship under normal use and service. This warranty includes all mechanical and electrical components.

Aluminium structural components and mechanical components on lifts are covered under warranty for a period of five (5) years from the date of delivery.

Damage caused by use in unsuitable environmental conditions or failure to maintain the product in accordance with user and service instructions is not covered. Any alteration, modification or repair unless performed by or authorized in writing by Joerns Healthcare will void this warranty.

### Parts:

HoyerPro lifts contain various parts that wear from normal use. These parts, such as DC batteries and casters are not covered under the two-year warranty but are covered for 90 days after date of delivery.

Joerns Healthcare's obligation under this warranty is limited to supplying replacement parts, servicing or replacing, at its option, any product which is found by Joerns Healthcare to be defective.

Warranty replacement parts are covered by the terms of this warranty until the product's original warranty period expires.

When requested by Joerns Healthcare, parts must be returned for inspection at the customer's expense. Credit will be issued only after inspection.

### Service:

Most service requests handled by the facility Maintenance Department with assistance from the Joerns Healthcare Product Service Department.

Most parts requested can be shipped next day air at the customer's expense.

Should a technician be required, one will be provided by Joerns Healthcare, at our discretion. Only the Joerns Healthcare Product Service Department can dispatch authorized technicians.

This warranty is extended to the original purchaser of the equipment.

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